

CLEANING INSTRUCTIONS ON EASYWIPE™ PU COATED MATTRESS PROTECTORS



PRODUCT NAME	ACTIVE COMPONENT	CONC.	RESULT (Short Contact)	RESULT (Long Contact)
Chlor-Clean Tablets	troclosene sodium anhydrous	1000 ppm (1 tablet / 1 liter water)	OK	OK (possibly discolor of the PU)
Dettol	chloroxylenol	5%	OK	OK
Ethanol 70%	ethanol	70%	OK	OK (material starts to wrinkle+resistance PU-surface slightly decreased)
Formaldehyde	-	30%	OK	OK
Hansamed (spray)	chlorhexidin-digluconas 1%	pure	OK	OK
Haz-Tabs	natriumdichloride	1000 ppm	OK	OK
Haz-Tabs	natriumdichloride	10.000 ppm	OK	not OK (discolor of the PU)
Hibicet	chlorhexidin-digluconate	pure	OK	OK
Incidin Plus	glocoprotamine	2%	OK	OK
Isopropylalcohol	isopropanol 70%	pure	not OK (material starts to wrinkle+resistance PU-surface decreased)	not OK (material starts to wrinkle+resistance PU-surface decreased)
Jodtinktur	jodium-kaliumjodine-ethanol	pure	not OK (discolor of the PU)	not OK (discolor of the PU)

Kodan Tinktur Forte	hydroperoxide-solution 30%	pure	OK (resistance PU-surface slightly decreased)	OK (resistance PU-surface slightly decreased)
Natriumhypochlorite	active chlorine	3%	not OK (discolor of the PU+resistance PU-surface decreased)	not OK (discolor of the PU+resistance PU-surface heavily decreased)
		0.3%	OK (resistance PU-surface slightly decreased)	not OK (discolor of the PU+resistance PU-surface decreased)
Octenisept (spray)	-	pure	OK	OK (resistance PU-surface slightly decreased)
P3-Topaxx 66	sodium hypochloride + hydroxide	5%	OK	OK (material starts to wrinkle+possibly discolor of the PU))
Sagrotan (spray)	-	pure	OK	OK
Sanit P20	mono-ethanol amina	pure	OK	OK
Sagrotan Med (spray)	alkylhydroxypolyoxylene	pure	OK (material starts to wrinkle)	OK (material starts to wrinkle+resistance PU-surface slightly decreased)
Softasept N (spray)	ethanol-propanol	pure	OK (material starts to wrinkle)	OK (material starts to wrinkle)
Spring	glycoether-alcohol	pure	OK	OK
Sterillum	glycoether 85% etradecanol	pure	OK (material starts to wrinkle)	OK (material starts to wrinkle+resistance PU-surface slightly decreased)
Suma Bac D10	ammoniumchloride	1%	OK	OK
Surfa'Safe	didecyldimethylamoniumchloride	pure	OK	OK
Topaxx 421	sodium hypochloride +hydroxide	5%	OK	OK (material starts to wrinkle)
Virkon	potassium prexomonosulphate	10g/l	OK	OK
Volvone	ammonia	40%	OK	OK